

Travel Guide

XPO₂™



User: BEFORE traveling with this product, read this manual and save for future reference.

For more information regarding Invacare products, parts, and services, please visit www.invacare.com



Yes, you can.®

Traveling with the XPO₂

- To ensure safe travel with the XPO₂, these instructions must be followed.
- For all warnings and cautions pertaining to the use of the XPO₂, please refer to the owner's manual.
- **No smoking or open flame is permitted within 10 feet (3 m) of where you are sitting.**
- **A spontaneous and violent ignition may occur if oil, grease, greasy substances, or petroleum based products come in contact with oxygen under pressure. These substances MUST be kept away from the XPO₂ portable concentrator, tubing and connections, and all other oxygen equipment. DO NOT use any lubricants unless recommended by Invacare.**

Traveling by Air

Before Traveling

1. Check www.invacare.com for a complete list of airlines that have approved the use of the XPO₂ on board.
2. Contact your travel carrier and/or tour guide about your travel plans and provide information about your intent to use a portable oxygen concentrator (POC) during your travels. The carrier will let you know if you will be permitted to use your XPO₂ during your travel and if there are any restrictions for use.

NOTE: If you are not permitted to use your XPO₂ on the airline of choice and still want to take it with you:

- Carry it and any supplemental batteries on board with you.
 - Ensure the unit is turned off.
 - Stow it properly for the flight.
 - **DO NOT place the XPO₂ or any supplemental battery modules in your checked baggage.**
3. Contact the airline to see if they have seats with a power outlet that you can use to power the XPO₂ during the flight. If a power outlet is not available, be sure to bring enough supplemental batteries to last 150% of the flight time.

There may be unexpected delays beyond scheduled flight times. Carry your power adapters with you so you can recharge batteries before and after the flight.

4. Ensure you have a licensed physician's signed statement on a standard form (p/n 1154266), the doctor's letterhead or an airline-approved form that contains the following information:

- Your need for oxygen during taxi, takeoff, flight times, landing, or the entire time you are traveling.
- The flowrate(s) that you will use during your trip.
- Your understanding, or that of your travel companion, of all the cautions, warnings, and alarms of the XPO₂ and what to do if there is an alarm situation with the device.
- Your personal information such as name, date of birth, and the model name or number of the portable concentrator.

You MUST have this form with you at all times during your travel.

5. Charge the internal battery of the XPO₂ and any supplemental batteries fully before you depart, thus extending the operating time of the XPO₂ during travel. Refer to [Recharging the Battery](#) in the owner's manual.
6. Ensure the unit is free of all grease, oil, or other petroleum products and that the unit is in good working order, is free of damage, and the air filter is clean. Refer to [Maintenance](#) in the owner's manual.
7. Contact your oxygen supplier to make arrangements if you need to have backup oxygen at your destination.

Arriving at the Airport

1. Allow the security agent to inspect your XPO₂, even if you are using it, when going through the security checkpoint.
They are required to allow you to travel through the checkpoint with your POC, but they MUST inspect it for security reasons.
2. Use your AC power adapter while you are in the airport, if possible. This will keep the batteries at full charge and give you the most battery powered operating time on the aircraft.

Boarding the Plane

You are not permitted to sit in an exit row if you plan to use your XPO₂ at any time during the flight.

1. If you plan on using the XPO₂ the entire time you are the plane, verify that your XPO₂ does not block access to an emergency exit or aisle way. If it does, request a seat change.
2. Show the pilot or flight crew your signed physician's statement regarding your use of the XPO₂, if asked.
3. In order for the flight crew to ensure the safety of you and all of the other passengers on the flight, allow them to inspect the XPO₂ to confirm that it is FAA approved for use on the flight.
4. If you are not planning to use your XPO₂ during taxi, takeoff, or landing, you **MUST** store it in an approved storage area so it does not block the row or aisle access.

NOTE: Contact the flight crew about location of approved storage areas.

On the Plane

You MUST store the supplemental battery either securely connected to the XPO₂ or in your carry-on baggage. The supplemental battery MUST be protected from damage and shorting out the external power connector.

1. You may use your XPO₂ when moving about the cabin, **ONLY** after the pilot turns off the "Fasten Seat Belt" sign.
2. Turn off the unit if the XPO₂ alarms during the flight, unless the alarm resets itself. If the alarm does not reset, the XPO₂ **MUST** be turned off and stored in an approved storage area.
3. If an AC power outlet is available on the flight, perform the following:
 - A. Connect the power adapter to the XPO₂.
 - B. Plug in the power adapter into the airline power outlet.
 - C. Turn the unit on.

NOTE: If you have any trouble making the connections to the plane's power outlet, contact the flight crew for assistance.

After the Flight

1. Ensure you have enough power to run your XPO₂ while leaving the airport. Recharge the batteries, if necessary. Refer to Recharging the Battery in the owner's manual.
2. Contact your medical gas supplier to obtain your backup oxygen supply, if necessary.

Traveling by Boat

Contact the cruise line and inform them you are traveling with a POC. There should be no restrictions on your use of the XPO₂ during your travel but it is wise to check with them ahead of time. Power should be available for use of your AC power adapter during your travels, but check first.

Traveling by Train

Contact the train authorities at least twelve hours ahead of arrival that you are going to use your POC. There should be no restrictions on your use of the XPO₂, but power on the train may not be available for your use during travel times. Ensure there is enough battery life for your trip before you leave.

Traveling by Bus

Contact the bus line about using a POC. There should be no restrictions on your use of the XPO₂ during your trip, but power may not be available for your use during travel times. Ensure there is enough battery life for your trip before you leave.

Enjoy your trip and have a good time!




Invacare Corporation

www.invacare.com

USA

One Invacare Way
Elyria, Ohio USA
44036-2125
440-329-6000
800-333-6900
Technical Services
440-329-6593
800-832-4707

Invacare Corporation
2101 E. Lake Mary Blvd.
Sanford, FL 32773
407-321-5630

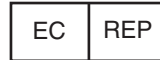

Canada

570 Matheson Blvd. E. Unit 8
Mississauga, Ontario, L4Z 4G4
905-890-8300
800-668-5324

EU Representative
Invacare International Sarl
Route de Cité Ouest 2
1196 Gland
Switzerland
Tel: +41 22 354 60 10
Fax: +41 22 354 60 11



All rights reserved. Trademarks are identified by the symbols [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted.
© 2008 Invacare Corporation